


THUASNE®

en	Sport lumbar support belt	4
fr	Ceinture de soutien lombaire sport	5
de	Flexible Sport-LWS-Stabilisierungsorthese	7
nl	Lumbale rugbrace sport	8
it	Cintura di sostegno lombare per uso sportivo	10
es	Faja de sujeción lumbar para uso deportivo	12
pt	Cinta de suporte lombar para uso desportivo	13
da	Lændestøttebælte sport	15
fi	Lannetukivyö urheilukäyttöön	16
sv	Stödbälte för ländryggen sport	18
el	Αθλητικη ζώνη οσφυϊκής υποστήριξης	19
cs	Vyztužený bederní pás sport	21
pl	Orteza lędzwiowa sportowa	22
lv	Sporta jostasvietas atbalsta josta	24
lt	Palaikomasis juosmens diržas sportui	26
et	Nimmeirkonna tugivöö sport	27
sl	Podporni ledveni pas šport	28
sk	Športový podporný driekový pás	30
hu	Ágyéki fűző sportoláshoz	31
bg	Колан за поддържане на кръста за спорт	33
ro	Centură de susținere lombară sport	35
hr	Lumbalni potporni pojas sport	36

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Oδηγίες χρήσης - Pokyny k používání - Szczegółły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu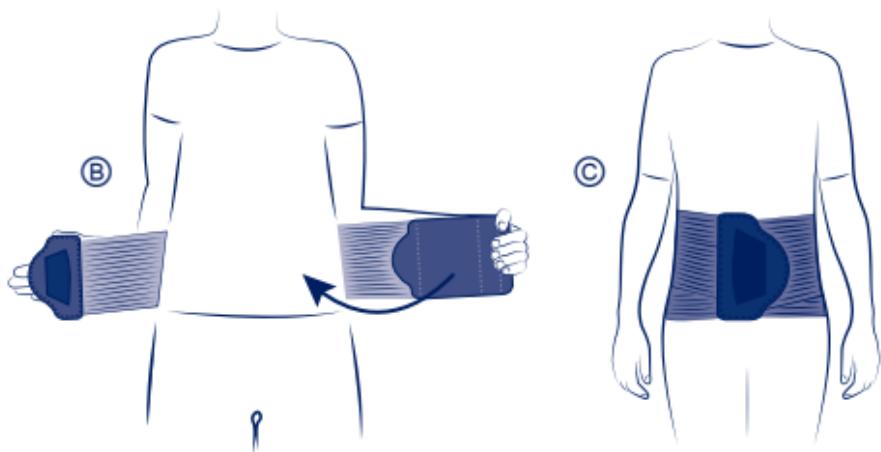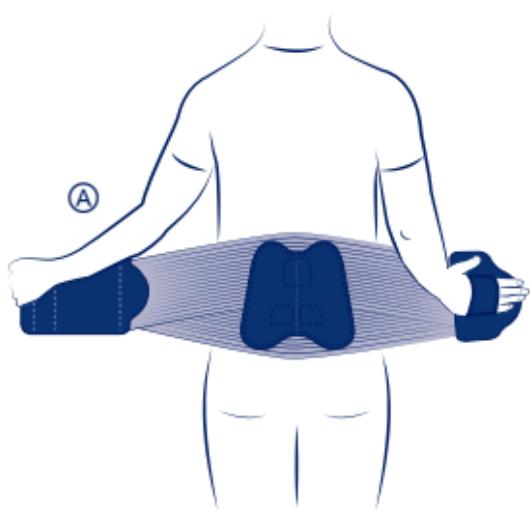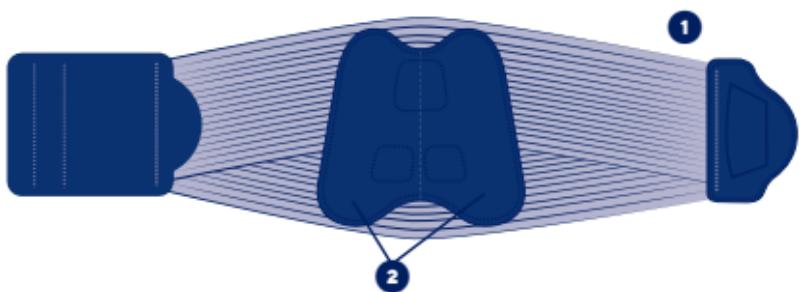

	cm
S	60 - 70
M	71 - 86
L	87 - 102
XL	103 - 120
XXL	121 - 140

en	Waist circumference
fr	Tour de taille
de	Taillenumfang
nl	Tailleomvang
it	Circonferenza vita
es	Contorno de cintura
pt	Perímetro de cintura
da	Taljemål
fi	Vyötärönympärys
sv	Midjemått
el	Περίμετρος μέσης
cs	Obvod pasu
pl	Obwód w talii
lv	Vidukļa apkārtmērs
lt	Juosmens apimtis
et	Vööümbermõõt
sl	Obseg pasu
sk	Obvod pása
hu	Derékbőség
bg	Обиколка на талията
ro	Circumferința taliei
hr	Opseg struka

MD

en

SPORT LUMBAR SUPPORT BELT

Support level 3

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

This device can be used when resuming sports activities.

Composition

Elastic fabric ①

Conformable dorsal stays ②

Textile components: polyamide - polyurethane - polyester - elastane.

Non-textile components: polyamide (self-fastening hook) - brushed tempered steel (dorsal stays) - polyoxymethylene (ventral stays).

Properties/Mode of action

The device relieves low back pain, supporting the lumbar spine: compartmental lumbar unloading is therefore achieved via increased intra-abdominal pressure.

Indications

Non-specific low back pain (acute, subacute and chronic phases).

Contraindications

Do not use on pregnant women.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of a hiatus hernia.

Do not use in the event of bone cancer with spinal metastases.

Do not use in the event of circulatory, pulmonary or cardiovascular problems in patients for whom an increase in arterial blood pressure is not recommended.

Do not use the product if the diagnosis has not been confirmed.

Precautions

Choose the appropriate size to fit the patient, referring to the size chart.

Verify the product's integrity before every use.

Wear the device above thin clothing.

Do not use the device if it is damaged.

Do not wear the device in a medical imaging machine.

Do not wear the device while sleeping.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

For hygiene, security and performance reasons, do not re-use the product for another patient.

The use of a medical device by a child should be done under the supervision of an adult or a healthcare professional.

Undesirable side-effect

This device can cause skin reactions (redness, itching, irritations...).

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Shape the dorsal stays to the curvature of the back by hand before first use.

Position the belt around the waist, using the finger loop and holding the left panel in the left hand. ④

The belt must be centred on the spine: the dorsal stays should be positioned on either side of the spine.

The bottom of the belt must stop at the top of the crease between the buttocks.

The Thuasne® logo indicates which way up the belt should be put on.

Pull the finger loop and the left panel simultaneously and evenly.

Fasten the belt, right side over left side, adjusting to the desired compression. ⑤ ⑥

The level of compression can be adjusted as required throughout the day.

Garment care

Close the self-fastening tabs before washing. Machine washable at 30 °C (delicate programme). Do not machine wash the device more than 10 times. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not tumble-dry. Do not iron. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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Fermer la ceinture côté droit par-dessus le côté gauche, en ajustant le niveau de serrage désiré. ® ©
THUASNE UK Ltd**
Thuasne SAS - SIREN/RCS Nanterre 542 091 186