


**THUASNE®**

<b>en</b>	<b>Novelastic sport strapping ankle brace .....</b>	<b>4</b>
<b>fr</b>	<b>Chevillère strapping Novelastic sport .....</b>	<b>5</b>
<b>de</b>	<b>Novelastic Strapping Sport-Knöchelbandage .....</b>	<b>6</b>
<b>nl</b>	<b>Novelastic enkelbrace met elastische banden sport .....</b>	<b>8</b>
<b>it</b>	<b>Cavigliera strapping Novelastic per uso sportivo .....</b>	<b>9</b>
<b>es</b>	<b>Tobillera strapping Novelastic para uso deportivo .....</b>	<b>11</b>
<b>pt</b>	<b>Tornozeleira de "strapping" Novelastic para uso desportivo .....</b>	<b>12</b>
<b>da</b>	<b>Ankelbandage Novelastic sport .....</b>	<b>14</b>
<b>fi</b>	<b>Nauhallinen Novelastic-nilkkatuki urheilukäytöön .....</b>	<b>15</b>
<b>sv</b>	<b>Ankelortos med elastiska band Novelastic sport .....</b>	<b>16</b>
<b>el</b>	<b>Αθλητική επιστραγαλίδα strapping Novelastic .....</b>	<b>18</b>
<b>cs</b>	<b>Pásková kotníková bandáž Novelastic sport .....</b>	<b>19</b>
<b>pl</b>	<b>Bandażowa orteza skokowo-stopowa Novelastic sportowa .....</b>	<b>21</b>
<b>lv</b>	<b>Novelastic sporta potītes ortoze .....</b>	<b>22</b>
<b>lt</b>	<b>Apjuosiamas kulkšnies įtvaras „Novelastic“ sportui .....</b>	<b>23</b>
<b>et</b>	<b>Novelastic hüppeliigese sporditugiside .....</b>	<b>25</b>
<b>sl</b>	<b>Opornica iz trakov za gleženj Novelastic šport .....</b>	<b>26</b>
<b>sk</b>	<b>Športová ortéza členka s páskami Novelastic .....</b>	<b>28</b>
<b>hu</b>	<b>Novelastic pántos bokarögzítő sportoláshoz .....</b>	<b>29</b>
<b>bg</b>	<b>Ортеза за глезнен с каишки Novelastic за спорт .....</b>	<b>30</b>
<b>ro</b>	<b>Gleznieră cu benzi de conțenie Novelastic sport .....</b>	<b>32</b>
<b>hr</b>	<b>Sportski steznik za gležanj s trakom Novelastic .....</b>	<b>33</b>

## **SPORT**

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Oδηγίες χρήσης - Pokyny k používání - Szczegółły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu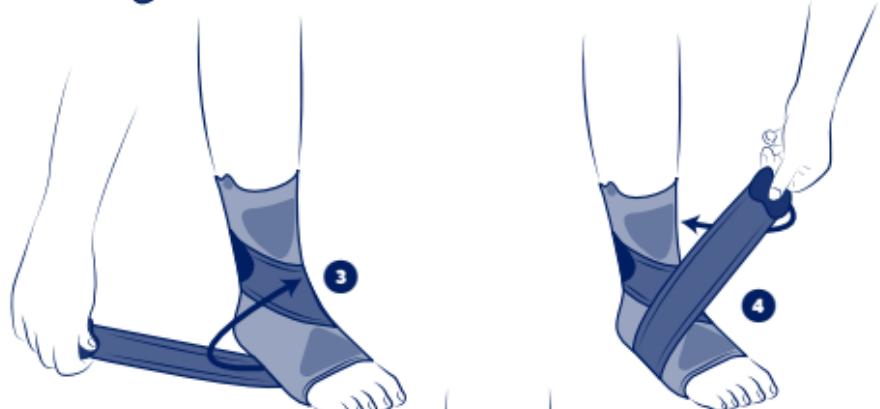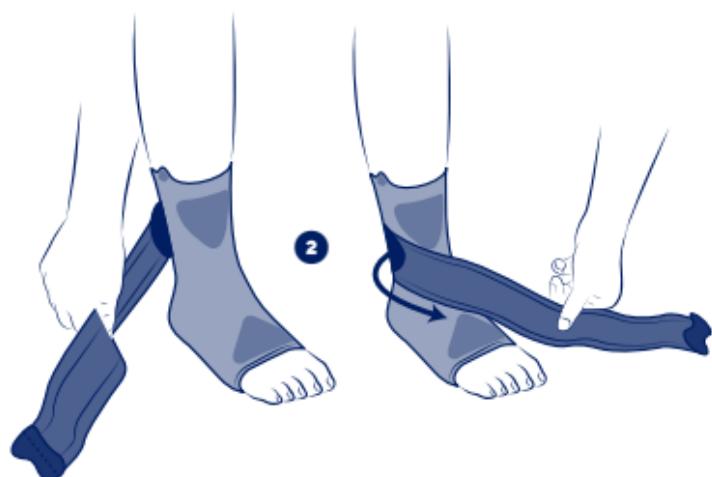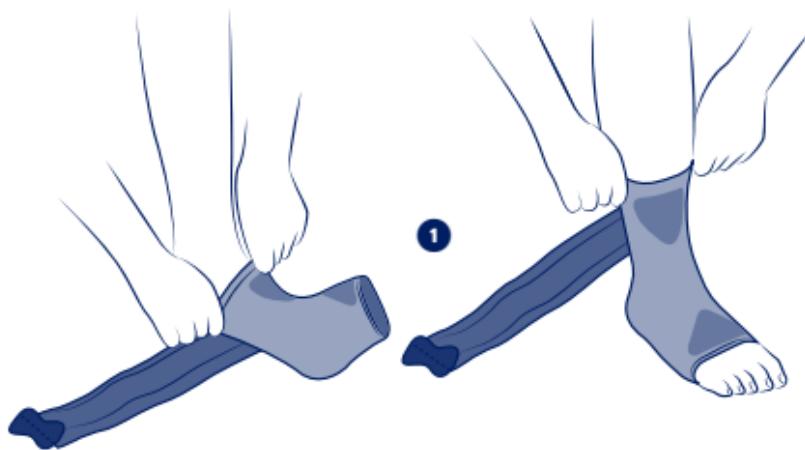

	cm
<b>S</b>	19 - 21
<b>M</b>	21 - 23
<b>L</b>	23 - 25
<b>XL</b>	25 - 27
<b>XXL</b>	27 - 30

<b>en</b>	Ankle circumference
<b>fr</b>	Tour de cheville
<b>de</b>	Knöchelumfang
<b>nl</b>	Omvang van de enkel
<b>it</b>	Giro caviglia
<b>es</b>	Circunferencia del tobillo
<b>pt</b>	Perímetro do tornozelo
<b>da</b>	Anklens omkreds
<b>fi</b>	Nilkanypärys
<b>sv</b>	Runt fotleden
<b>el</b>	Περίμετρος αστραγάλου
<b>cs</b>	Obvod kotníku
<b>pl</b>	Obwód kostki
<b>lv</b>	Potītes apkārtmērs
<b>lt</b>	Kulkšnies apimtis
<b>et</b>	Pahkluu ümbermõõt
<b>sl</b>	Obseg gležnja
<b>sk</b>	Obvod členka
<b>hu</b>	Boka kerülete
<b>bg</b>	Обиколка на глезена
<b>ro</b>	Circumferința gleznei
<b>hr</b>	Opseg gležnja

**MD**

# NOVELASTIC SPORT STRAPPING ANKLE

## BRACE

*Support level 3*

### Description/Intended use

This device is a proprioceptive elastic ankle brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. This device can be used as a proprioceptive support for rehabilitation, particularly when resuming sporting activities.

### Composition

Polyamide - elastodiene - elastane - polyurethane - polyester.

### Properties/Mode of action

Elastic compressive knitting.

Comfort zone over the ankle bones.

### Indications

Joint instability/laxity.

Mild ankle sprain (acute phase).

### Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

### Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended to adequately tighten the device to achieve support without restricting blood circulation.

Consult a healthcare professional in case of uncertain diagnosis.

Consult a healthcare professional and stop using the device:

- in case of discomfort, significant hindrance, pain, abnormal sensations,
- In case of changes in the device's performance.

In case of doubt before any sports activity, consult a healthcare professional.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

### Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### Instructions for use/Application

Put on the ankle support (logo facing up) positioning the heel in the appropriate area and limiting the extension strap to the rear of the foot. ①

Take the strap on the outer side of the foot and run it over the instep towards the inside. ②

Pass the strap under the foot from the inside to the outside. ③  
Pass the strap over the instep and secure it inside the ankle. ④

### **Garment care**

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Machine washable at 30 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

### **Storage**

Store at room temperature, preferably in the original packaging.

### **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2050701 (2023-12)
THUASNE UK Ltd**
Thuasne SAS - SIREN/RCS Nanterre 542 091 186