
THUASNE®

en	Sport reinforced ankle brace	4
fr	Chevillère de maintien renforcée sport	5
de	Verstärkte Sport-Knöchelbandage	6
nl	Verstevigde steunbrace voor de enkel sport	8
it	Cavigliera di sostegno rinforzata per uso sportivo	9
es	Tobillera de sujeción reforzada para uso deportivo	10
pt	Tornoeleira de suporte reforçada para uso desportivo	12
da	Forstærket ankelstøttebind sport	13
fi	Vahvistettu nilkkatuki urheilukäyttöön	14
sv	Förstärkt ankelortos för stöd sport	16
el	Αθλητική ενισχυμένη επιστραγαλίδα συγκράτησης	17
cs	Zesílená kotníková bandáž sport	18
pl	Wzmocniona podtrzymująca orteza skokowo-stopowa sportowa	20
lv	Sporta potītes atbalsts ar pastiprinājumu	21
lt	Sutvirtintas palaikomasis kulkšnies įtvaras sportui	22
et	Hüppeliigese tugevdatud sporditugiside	24
sl	Ojačana opornica za gleženj šport	25
sk	Športová vystužená ortéza členka	26
hu	Megerősített támasztó bokarögzítő sportoláshoz	27
bg	Подсилена поддържаща ортеза за глезен за спорт	29
ro	Glezneră de susținere întărită sport	30
hr	Sportski ojačani steznik za potporu gležnja	32

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu

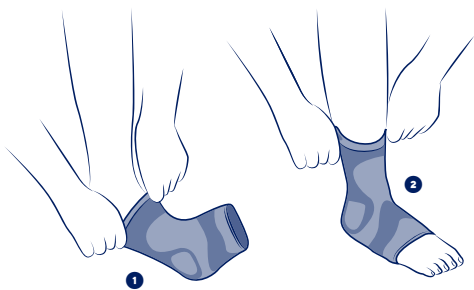cm
S
19 - 21
M
21 - 23
L
23 - 25
XL
25 - 27
XXL
27 - 30

en	Ankle circumference
fr	Tour de cheville
de	Knöchelumfang
nl	Omvang van de enkel
it	Giro caviglia
es	Circunferencia del tobillo
pt	Perímetro do tornozelo
da	Anklens omkreds
fi	Nilkanympärys
sv	Runt fotleden
el	Περίμετρος αστραγάλου
cs	Obvod kotníku
pl	Obwód kostki
lv	Potītes apkārtmērs
lt	Kulkšnies apimtis
et	Pahkluu ümbermõõt
sl	Obseg gležnja
sk	Obvod členka
hu	Boka kerülete
bg	Обиколка на глезена
ro	Circumferința gleznei
hr	Opseg gležnja

en

SPORT REINFORCED ANKLE BRACE

Support level 2

Description/Intended use

This device is a proprioceptive elastic ankle brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. This device can be used as a proprioceptive support for rehabilitation, particularly when resuming sporting activities.

Composition

Polyamide - elastodiene - elastane - viscose - natural latex.

Properties/Mode of action

Elastic compressive knitting.

Comfort zone over the ankle bones.

Indications

Joint instability/laxity.

Mild ankle sprain (acute phase).

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

Consult a healthcare professional in case of uncertain diagnosis.

Consult a healthcare professional and stop using the device:

- In case of discomfort, significant hindrance, pain, abnormal sensations,
- In case of changes in the device's performance.

In case of doubt before any sports activity, consult a healthcare professional.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

Contains natural latex.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Pull the ankle brace over the ankle joint, like a sock. ❶

Place the malleoli correctly, on the comfort zones. ❷

Garment care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Hand wash. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2050401 (2023-12)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186